

Kodak InSight

Dental Film

Minimize exposure. Maximize results.

The standard for intraoral film. With Kodak InSight dental film, you'll get a true F-speed film that requires up to 60% less radiation per exposure.



- Delivers excellent and consistent image quality through exclusive T-Grain® emulsion technology.
- Lowers patient and staff radiation exposure without sacrificing image resolution.
- Needs no continual equipment adjustments to accommodate processing conditions.
- Holds speed and contrast in variable chemistry conditions.
- Is available in all intraoral film sizes and formats, including ClinAsept barrier packets for added safety and SureSoft packets for added patient comfort.

Exposure guidelines

KODAK ULTRA-SPEED Dental Film users		E-Speed Film users	
Automatic processing	Reduce exposure time 60%*	Automatic processing	Reduce exposure time 20%*
Manual processing	Reduce exposure time 50%*	Manual processing	Reduce exposure time 20%*

For best results, use with Kodak Readyomatic, Readypro, or GBX processing chemicals.

*Reduce exposure time only. Do not change mA or kVp settings.

Kodak InSight dental film processes as an F-speed film when processed in roller transport automatic processors; on the high-end of E-speed range when processed manually.

Ordering Information

Size	Description	Code	Films/Pkgs	CAT Number
Size 0	Kodak Super Poly-Soft packets	IP-01	100	867 5332
Size 0	Kodak Super Poly-Soft packets (double film)	IP-02	100	128 0619
Size 0	Kodak Super Poly-Soft packets with ClinAsept barrier	IP-01C	75	120 0328
Size 0	Kodak Super Poly-Soft packets with ClinAsept barrier (double film)	IP-02C	75	124 7410
Size 0	Kodak bite-wing paper packets	IB-01	50	811 0785
Size 1	Kodak paper packets	IP-11	100	112 4981
Size 1	Kodak paper packets (double film)	IP-12	100	805 5402
Size 2	Kodak InSight dental film in SureSoft packets	IP-21S	80	811 8226
Size 2	Kodak Super Poly-Soft packets	IP-21	150	116 3401
Size 2	Kodak Super Poly-Soft packets (double film)	IP-22	130	179 8628
Size 2	Kodak Super Poly-Soft packets with ClinAsept barrier	IP-21C	100	153 9931
Size 2	Kodak Super Poly-Soft packets with ClinAsept barrier (double film)	IP-22C	100	156 0390
Size 2	Kodak paper packets	IP-21	150	176 3960
Size 2	Kodak paper packets (double film)	IP-22	150	107 9086
Size 2	Kodak bite-wing paper packets	IB-21	50	180 7650
Size 3	Kodak bite-wing paper packets	IB-31	100	829 8929
Size 4	Kodak occlusal paper packets	IO-41	25	116 9143
Size 4	Kodak occlusal paper packets (double film)	IO-42	25	819 5240

For more information, call **1-800-933-8031** or visit: www.kodakdental.com



Carestream Health, Inc.

©Carestream Health, Inc., 2009.
InSight, Ultra-speed, SureSoft, Super Poly-Soft, ClinAsept, Readyomatic and Readypro are trademarks of Carestream Health.
The Kodak trademark and trade dress are used under license from Kodak.