

Kodak T-Mat G/RA

Dental Film

Kodak T-Mat G/RA dental film is an extraoral x-ray film for use with green-emitting screens.

When paired with Lanex regular or medium screens, it is an excellent film-screen combination for panoramic, cephalometric, TMJ, bilmolar or carpal bone radiography.



T-Mat G/RA dental film is a high-contrast film with excellent image detail. It is extremely stable and its sensitometric characteristics provide increased processing latitude and resistance to process and handling artifacts.

Processing

Automatic

Kodak T-Mat G/RA dental film can be processed in a variety of automatic processors. Recommended chemistry for standard dental processors is Kodak Readymatic dental developer and fixer.

Manual

Kodak T-Mat G/RA dental film can also be processed manually. Recommended chemicals are Kodak GBX developer and fixer.

Description	Temperature	Time	Agitation
Developer	20°C	5 min	only at the beginning to remove any surface bubbles
Stopbath (water)	16 to 30°C	30 sec	continuously
Fixer	16 to 30°C	3-5 min	continuously at the beginning then every 30 seconds
Rinsing	16 to 30°C	5-10 min	running water
Drying	max 50°C		

Exposure Guidelines for T-Mat G/RA dental film for adult patients* used with Kodak Lanex regular screens.

Examination	Image Receptor Distance	Approximate Exposure in Seconds**		
		65 kVp 10 mA	75 kVp 10 mA	90 kVp 10 mA
Ramus of mandible; body of mandible	38 cm (15 in)	1/15	1/20	1/30
Lateral skull	76 cm (30 in)	3/5	3/10	1/12
Posteroanterior skull (granger technic)	76 cm (30 in)	2/3	1/3	1/5
Temporomandibular articulation	25.4 cm (10 in)	1/10	1/20	1/30
Cephalometric radiography	152 cm (60 in)	2/3	1/3	1/5

*For children: Reduce adult exposure time about one-third to one-half.

**Based on half-wave rectified units.

Ordering Information*

Size	Description	Films/Pkgs	CAT Number
Panoramic radiography			
12.7 x 30.5 (5 in x 12 in)	T-Mat G/RA	50	198 7627
12.7 x 30.5 (5 in x 12 in)	T-Mat G/RA	100	521 5637
15.0 x 30.0 (6 in x 12 in)	T-Mat G/RA	50	186 9080
15.0 x 30.0 (6 in x 12 in)	T-Mat G/RA	100	521 5603
12.7 x 30.5 (5 in x 12 in)	T-Mat G/RA panoclass**	100	521 5645
15.0 x 30.0 (6 in x 12 in)	T-Mat G/RA panoclass**	100	521 5611
Cephalometric radiography			
18.0 x 24.0	T-Mat G/RA	50	521 2022
20.3 x 25.4 (8 in x 10 in)	T-Mat G/RA	50	845 3938
Bimolar, lateral skull and carpal bone radiography			
12.7 x 17.8 (5 in x 7 in)	T-Mat G/RA	50	146 4957

*Check with your local distributor for availability of specific items

**With perforations to allow film to be folded for easier fit in archive records

For more information, call **1-800-933-8031** or visit: www.kodakdental.com



Carestream Health, Inc.

©Carestream Health, Inc., 2009.

Lanex, T-Mat and Readymatic are trademarks of Carestream Health.

The Kodak trademark and trade dress are used under license from Kodak.